

LCD explanations

Depending on your study, you may be using MEMS monitors with LCD screen.

The LCD shows:

- how many times the bottle has been opened each day.
- how many hours have passed since the bottle was last opened.



Example

- The number 1 shows that the bottle has been opened once today.
- The 3 dashes show that 3 hours have passed since the bottle was opened.
- The next time the bottle is opened, the number 1 will change to 2 and the dashes will disappear.



Example

- When more than 12 hours have passed since the bottle was opened, the dashes begin to blink.
- The picture shows that 13 hours have passed since the bottle was opened last.

Notes

- The LCD resets to zero each night.
- If the bottle is not opened for more than 1 week, the LCD display will go to sleep and the LCD screen will be blank. As soon as the bottle is opened again, the display will automatically turn on.
- An interval of at least 3 seconds between opening and subsequent closure of the medication bottle is necessary to record the opening.