



*Instructions for the patients*

---



The MEMS cap will be used in the clinical study to record the date and time when you take your medication.



Only open the bottle when taking your medication and close it with the MEMS Cap afterwards (push and turn if child resistant).



Bring your bottle(s) and MEMS Cap to all your appointments



Always keep your medicine in the bottle with the MEMS Cap.

**DO NOT use a weekly organizer or any other pillbox!**



Always keep the bottle closed with the MEMS cap.



**DO NOT throw away the MEMS cap!**  
**DO NOT put the MEMS cap in the water!**