


View patient data

Depending on your profile, after the reading of your MEMS caps, you will not see any patient's adherence data. However, you are allowed to comment on the patient's MEMS use.

1. After the reading of the MEMS caps, you will be directed to the following screen:

a. You may enter any comment regarding the patient's adherence. To do so, click on +Add comment and save the form..

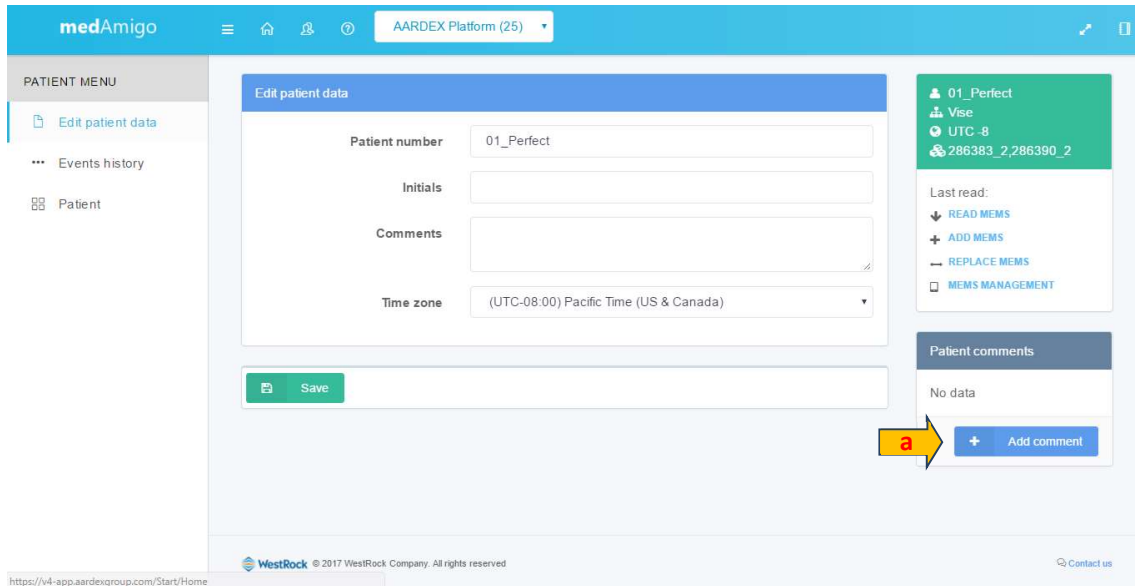

The screenshot displays the 'medAmigo' interface for editing patient data. The main content area is titled 'Edit patient data' and contains the following fields:

- Patient number: 01_Perfect
- Initials: (empty)
- Comments: (empty text area)
- Time zone: (UTC-08:00) Pacific Time (US & Canada)

A green 'Save' button is located at the bottom left of the form. On the right side, a patient summary card for '01_Perfect' is visible, showing 'Vise', 'UTC -8', and '286383_2,286390_2'. Below the summary card, there are options: 'READ MEMS', '+ ADD MEMS', '- REPLACE MEMS', and 'MEMS MANAGEMENT'. At the bottom right, there is a 'Patient comments' section with 'No data' and a '+ Add comment' button. A red arrow labeled 'a' points to the '+ Add comment' button.